SIDEXIS 4 PC CHECK

(For English version see page 3)

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SIDEXIS 4 PC CHECK

SIDEXIS 4 PC CHECK is a software, which checks, if...


- ...a workstation fits to the system requirements of SIDEXIS 4
- ...the units and software components in the installation environment are compatible with SIDEXIS 4

Please note: SIDEXIS 4 PC CHECK does not check, whether units and software components are compatible with this computer.

How to use SIDEXIS 4 PC CHECK:


There is no need to install SIDEXIS 4 PC CHECK. Simply copy the folder „SIDEXIS 4 PC CHECK“ to a shared drive and start the file „SIDEXIS4_PCCheck.exe“ by a double-click. Alternatively, you can run the software out of an USB drive.

After starting, the main menu with three steps opens: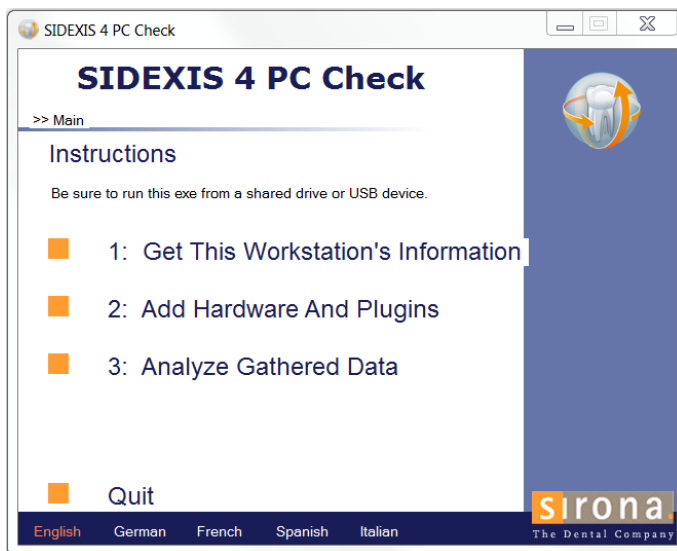


First, click on „1: Get this workstation’s information“. The software will now collect the necessary information from the computer. This will only take a few seconds and is finished as soon as a green hook appears.

Second, please click on „2: Add hardware and plugins“.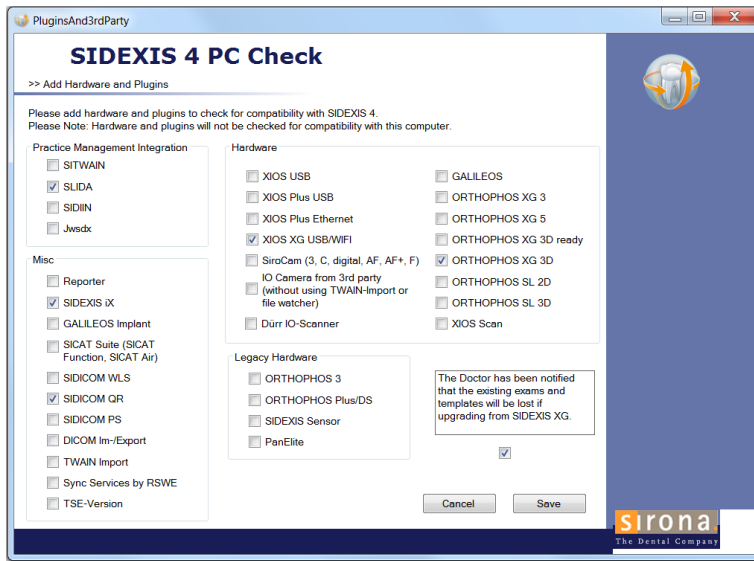

Please choose all components, which you would like to check for compatibility with SIDEXIS 4. Click “Save” after choosing.

Third, please click on „3: Analyze Gathered Data“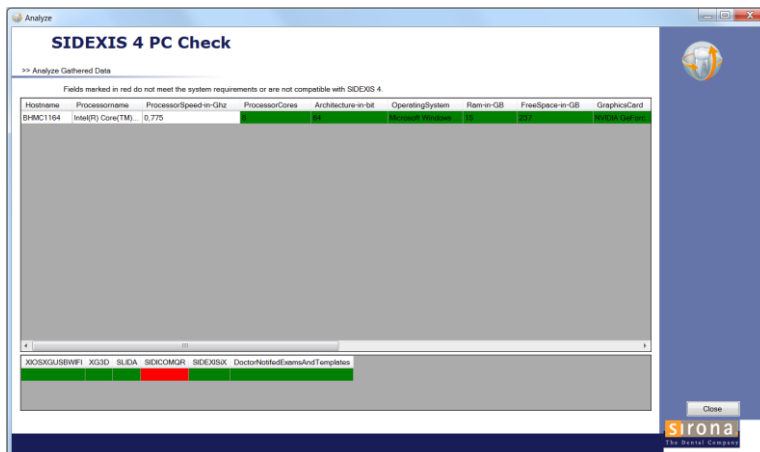

At the top you may find a table stating all system requirements of SIDEXIS 4 (horizontal scrolling might be necessary). A green mark means, that a component fits to the system requirements and a red mark shows that the requirements are not fulfilled in this point.

In the lower table you will find the components chosen in step 2. A green mark means that a component is compatible with SIDEXIS 4, a red mark means that this is not compatible.

Note: The information about the workstation collected in step 1 is saved in the file „WorkstationInformation.xml“. If you would like to reset SIDEXIS 4 PC CHECK, e.g. in order to check this workstation again, you can delete this file.